Hemoglobin A1c Analyzer

USER MANUAL

www.greencrossms.com

- Hemoglobin A1c Analyzer

Model.RT-100

Hemoglobin A1c Analyzer

1. IMPORTANT SAFETY INSTRUCTIONS

Make sure that you are fully aware of the manual before using the device.

- ① The device is for In Vitro Diagnostic use.
- ② Device is for medical professional use only.
- ③ Use GREENCARE A1c Hemoglobin A1c Test Kit only for the device.
- ④ Do not expose the device in overheated environment.
- ⑤ Do not expose the device in dusty or pollutant environment.
- (6) Do not use the device if it is not working properly, or if it has suffered any damage.
- ⑦ Conduct control solution test as directed in the manual.
- (8) The manual must be contained at proper place.

2. TABLE OF CONTENTS

1. Important safety instructions1p
2. Table of contents1p
3. Instruction1p
3-1. Intended use 1p
3-2. Operating of measurement 2p
4. Important information 2p
4-1. General precautions 2p
4-2. Product usage precautions 2p
4-3. Storage instruction 2p
4-4. Test instruction 2p
5. Contents of the device 3p
5-1. Contents 3p
5-2. Peripheral device(Option) 4p
6. Layout of product 4p
6-1. Device layout 4p
6-2. Screen layout 4p
6-3. Test Kit description 5p
6-4. Check Kit description 5p
7. Preparation before use 5p
7-1. Power on 5p
7-2. Printer paper checking 6p
7-3. HbA1c analyzer setting 6p
7-4. HbA1c analyzertest 8p

8. Control solution	9р
8-1. Control solution test	9р
9. Testing blood	12p
9-1. Preparation and storage method	12p
9-2. HbA1c test	13p
10. Test history	d15p
10-1. HbA1c test history	15p
10-2. Control solution result	16p
11 Managing glucose	17p
12 Transforring data	17p
12. Caring for the applyzor	17p
15. Calling for the analyzer	17p
13-1. Cleaning and disinfection	1/p
13-2. Storage and handing	17р
13-3. Checking the analyzer	18p
13-4. Error message and troubleshooting	19p
14. Specification	23p
15. Performance	23p
15-1. Accuracy	23p
15-2. Precision	23p
16. Warranty	24p
17 Symbols	25p
17.591115015	20p

3. INSTRUCTION

Thank you for your purchase. Please read the manual thoroughly to fully understand the instructions of the device before using it. Please keep the manual with the analyzer. If you have any questions, please contact your local representative.

3-1. Intended use

A dedicated device for measuring the concentration of glycated hemoglobin(HbA1c) present in human blood using methods such as HPLC, electrophoresis, or immunoassay.

Hemoglobin A1c Analyzer

3-2. Operating of measurement

This is a device for measuring hemoglobin A1c concentration in the blood by analyzing the intensity of light reflected from the measurement area, and it is an in vitro diagnostic device that is used professionally in hospitals and clinical laboratories.

After opening the analyzer cover, insert the cassette of the GREENCARE A1c Hemoglobin A1c Test Kit into the analyzer holder, collect blood from the blood sampling area at the end of the reagent pack, and insert the reagent pack into the cassette. When the reagent pack is inserted into the cassette, the sealed part on both sides of the reagent pack is released. The user closes the cover of the analyzer, the analyzer automatically starts the measurement operation. The result is displayed on the screen when the measurement is completed.

4. IMPORTANT INFORMATION

4-1. General precautions

- ① Before using the analyzer, be sure to read and follow the instruction given in the manual.
- ② It may damage the analyzer due to improper use and management, and may result in inaccurate measurement result.
- ③ When using the product, first check whether the meter is abnormal or not by using the supplied Check Kit.
- ④ If the instrument is moved or an error occurs, check the meter with the supplied Check Kit for use.

4-2. Product usage precautions

- ① If you are not going to use the analyzer, unplug it and store it.
- ② Use the supplied power adapter and power cable. If you use a power adapter and power cable that is not compatible with the analyzer, it may cause damage.
- ③ The screen is displayed, but if the analyzer does not work, turn off and on the power switch again to confirm operation. If it does not work, contact your local representative.
- ④ Do not leave the device where strong magnetic field.
- (5) Do not leave the device where humidity or temperature changes dramatically.
- ⑥ Use it on a flat surface without slope.
- ⑦ It the screen is unusual or if abnormal noise occur constantly, contact your local representative, please...
- ⑧ We are not responsible for damage caused by user's negligence or arbitrary disassembly.
- (9) To prevent cross contamination, wear gloves when using the product.
- 1 This analyzer is available only for GREENCARE A1c Hemoglobin A1c Test Kit .
- (1) Dispose of used Test Kit safely according to medical waste disposal procedures.
- ② The Analyzer must be disposed according to the local regulations concerning the disposal of electrical and electronic equipment. Contact our representatives in your area for the respective instructions before disposing.

4-3. Storage instruction

- ① The analyzer must be protected from the direct sunlight. It can change the color of analyzer.
- ② Do not leave the device at dusty area.
- ③ Store the device in the storage humidity and temperature.
- ④ Keep out of reach of unprofessional.

4-4. Test instruction

- ① Do not damage the cassette information label.
- ② Do not damage the cassette measurement window. Be careful not to make fingerprints on optical reading area of cassette.
- ③ When inserting the cassette into the cassette holder, insert it at once to ensure accurate recognition.

GREENCARE AIC Hemoglobin A1c Analyzer

- ④ Reagent pack reagent may agglutinate during storage for a long time, so shake well before inserting reagent pack.
- (5) If you need to remove the cassette due to an error when inserting the cassette, you can easily remove the cassette with the reagent pack.
 - Insert the reagent pack upside down and hook the protruding part of the reagent pack onto the cassette to remove the cassette.

5. CONTENTS OF THE DEVICE 5-1. Contents

- ① GREENCARE A1c analyzer
- ② Check Kit
- ③ User manual

- 4 Power adapter(include the AC Code)
- ⑤ Printer paper

Hemoglobin A1c Analyzer

(4) Control solution

(5) Test Kit

5-2. Peripheral device (Option)

- (1) USB cable for PC communication
- (2) Ethernet cable
- (3) Barcode scanner

6. LAYOUT OF PRODUCT

6-1. Device layout

- (1) Touch screen display: Touch Screen to select device menu. (5) USB connector 1: Connector to communicate PC.
- ② Thermal printer: The measurement result is printed.
- (3) Power switch: Power ON/OFF.
- ④ Power connector: Connector of power supply adapter.
- 34567 П
- (6) USB connector 2: Connector for barcode reader device.
- (7) LAN connector: Ethernet connector. (LIS connected)
- (8) Fan air hole: Hole of the air circulatory system.

6-2. Screen layout

- (1) Date: The present date is displayed.
- (2) Time : The present time is displayed.
- (3) Main area : The function of device and the measurement result are displayed.

Hemoglobin A1c Analyzer

6-3. Test Kit description

Reagent Pack Insert Area : The reagent pack is inserted in this area. Cassette Information : The available period and information of cassette. Measurement Window : The HbA1c is measured through this window. Reagent Storage : 1, 2 Reagent storage. Sampling Area : The blood is sampled in this area.

6-4. Check Kit description

Check Kit Information : The available period and information of Check Kit. **Measurement Window:** The HbA1c is measured through this window.

7. PREPARATION BEFORE USE

7-1. Power on

Connect the DC 12V power adapter to a grounded outlet, connect the power plug to the power port on the back side of analyzer, and check the operation.

After the analyzer turn on, the main screen is displayed. During the initialization screen display, it perform operations such as setting the data and setting values, setting the instrument and heating the heater temperature. If it has not been used for 10 minutes in the main screen, it changes to the power saving screen. If the power saving screen is displayed, touch the LCD to display the main screen.

Hemoglobin A1c Analyzer

7-2. Printer paper checking

If the printer is set to Use, open the printer cover with both hands and make sure that the printer paper is loaded correctly in the printer.

7-3. HbA1c analyzer setting

- Press " Settings" " button for setting menu.
 Press " multiple of the setting menu.

Hemoglobin A1c Analyzer

- 5) To set unit, press "unit " button in setting menu. Select "NGSP/IFCC" or "eAG" button, and press "ok " button.
- 6) To install the peripheral device, press " ting menu. Select "ON" or "OFF" for "Barcode Reader" and "Printer", and press " ok " button.
- 7) To set brightness, press " ^{Brightness} " button in setting menu. Move the icon in slide bar to set brightness, and press " <u>K</u>" button.

8) To check the outside temperature, press " () Temperature " button.

9) To set volume, press " volume " button in setting menu. Move the icon in slide bar to set volume, and press " or " button.

Hemoglobin A1c Analyzer

7-4. HbA1c analyzer test

10) To set language, press " Language " button in setting menu, Select the language, and press " OK " button.

- 3) There are Barcode Reader Test, Cassette Reader Test, Printer Test, Sensor Test, Motor Test and Error List in test menu.
- 4) To test a barcode reader, press " Barcode Reader " button in test menu. The barcode number is displayed if barcode reader reads cassette barcode.

www.greencrossms.com

Hemoglobin A1c Analyzer

6) To test a thermal printer operation, press "Printer Test " button in test menu. Press "Printer" button, and a test page is printed.

7) To test the sensors operation, press " C Sensor Test " button in test menu. You can test operation of a Cover Sensor, a Cassette Sensor, a Reagent Sensor. If "LED Sensor" is ON, LED blinked.

8) To test the motor operation, press " (Motor Test)" button in test

menu.

(Number 1 button : Clockwise 90 degrees rotation.

Number 2 button : Counterclockwise 90 degrees rotation.

Number 3 button : Clockwise, Counterclockwise 10 degrees rotation.)

9) To check of error list, press " S Error List "button in test menu. you can check for last 100 error list using " "and " " "buttons.

8. CONTROL SOLUTION 8-1. Control solution test

This test is used to check that the analyzer and Test Kit are operating normally. You can trust the HbA1c value of the sample only if the analyzer is checked periodically using a control solution and the test result is within the normal range. The GREENCARE A1c provides the Check Kit to determine if the analyzer is malfunctioning. If the analyzer may be something wrong, check with the Check Kit.

Perform a control solution test if

- You are using the HbA1c analyzer for the first time.
- You suspect the HbA1c analyzer or the Test Kit may not be functioning properly.
- The test results appear to be abnormally high or low.
- The Test Kit package was left open.
- You are using a new Test Kit package.
- You dropped the analyzer.

Hemoglobin A1c Analyzer

REFERENCE

The result of the control solution test is independent of the HbA1c of the user.

* The control solution is sold separately. If you would like to purchase, please contact your local distributor.

- 4) Select the level of control solution in control solution level mode, and press " οκ " button.
- 5) Select the range of control solution in control solution range mode, and press " οκ " button.
- 6) Shake the reagent pack well at 180 degrees to mix the reagents in the reagent pack. Gently mix well the reagent pack 5~6 times, and visually check that the beads are not lump. The mode is changed after 5 seconds.
- Apply the control solution sample at the sampling collecting area. The mode is changed after 5 seconds.
- 8) Insert the reagent pack into cassette which is placed in the cassette holder of the analyzer.
- 9) After the reagent pack is normally inserted, the "Close the cover" indication is displayed. Close the cover within 5 seconds. The test starts automatically once the cover is closed.

Hemoglobin A1c Analyzer

10) The rest of test time is displayed if the cover is closed.

11) After the test is done, the result value is displayed. Confirm the result value, press " ok " button. The test result value may be in measurement range.

12) Open the cover to remove the test cassette.

 Take out the test cassette from the analyzer cassette holder by pulling out.

9. TESTING BLOOD 9-1. Preparation and storage method.

Use fresh blood as sample.

1) Capillary

Wash your fingers well or wipe with alcohol swab, dry thoroughly, and then use a sterilized lancet to pierce the blood collection area and allow blood to come out. To gently draw large blood drops, gently press your fingers and wipe the first drop as it may contain tissue fluid. Press again with your hand gently to collect 3.5 µl of blood from the second blood drop to the blood collection area at the end of the reagent bottle.

2) Venous blood

Blood collected in venipuncture is collected in tubes treated with Sodium Heparin, K3EDTA, or NaF, an anticoagulant. The collected blood can be kept sealed at a temperature between $2 \sim 8$ °C for 1 week. If it is stored at $21 \sim 25$ °C, it can be kept sealed for 3 days. Drop one drop of blood into a dish on a clean plate and collect 3.5 µl of blood from the blood collection end of the reagent vessel.

Hemoglobin A1c Analyzer

9-2. HbA1c test

- 1) To measure HbA1c, press the cover button of the analyzer.
- 2) Insert the test cassette into the cassette holder when "Insert the test cassette" message is displayed. Please insert the cassette at a stroke.
 If it does not proceed to the next step, press the" multiple " button to proceed to the next step.
- 2-1) If the cassette code is not recognized correctly, a screen appears asking if you have used the test cassette. If you have used a test cassette, press the "Test Cassette?" button and " ок " button to proceed with the cassette code entry step.
- 2-2) Select the " `` or " `` or the test cassette code input screen, enter the test cassette code manually and press the " `` ok `` " button. The test cassette code can be found in the Test Kit box.
- 3) A barcode scanner when set to enable, the barcode scanner recognition screen appears. Once the ID is recognized by the barcode scanner, press " ok " button to proceed to the next step. If you do not have a barcode scanner, you can manually enter the ID by pressing the " input " button.
- 3-1) If you want to enter the ID manually, enter the ID and press the "OK" " button to proceed to the next step.

Hemoglobin A1c Analyzer

- 4) Shake the reagent pack well at 180 degrees to mix the reagents in the reagent pack. Gently mix well the reagent pack 5~6 times, and visually check that the beads are not lump. The mode is changed after 5 seconds.
- 5) Apply the blood sample at the sampling collecting area. This screen changes automatically after 5 seconds.
- 6) Insert the reagent pack into cassette which is placed in the cassette holder of the analyzer.
- 7) After the reagent pack is normally inserted, the "Close the cover" indication is displayed. Close the cover within 5 seconds. The test starts automatically once the cover is closed.

8) The rest of test time is displayed if the cover is closed.

9) After the test is done, the result value is displayed. Confirm the result value, press" or "button. The test result value may be in measurement range.

Hemoglobin A1c Analyzer

10) Open the cover to remove the test cassette.

11) Take out the test cassette from the analyzer cassette holder by pulling out.

10. TEST HISTORY 10-1. HbA1c test history

Please enter an ID to change		
	1 2 3 ABC	
ID SHINJUNGSUB123	4 5 6 🗲	
	7 8 9 0	
Cancel		

1) Press" Documents "button for HbA1c test history.

- 2) The test history list is displayed. Press either " " " " button to scroll up or down. After the test result is selected by pressing " " button, press "PRINT" button to reprint the selected result or " " " button to remove it. You can also change the ID by pressing the " ID " button.
- 2-1) You can change the ID by pressing the "ok" button after entering the ID on the ID input screen.

Hemoglobin A1c Analyzer

10-2. Control solution result

- 2-2) The result of selected ID is appeared, if the "ID" button is pressed, and a desired ID is selected.
- 2-3) Select "DATE" button, and press "OK" button. The results of selected date are displayed.

1) Press" Settings "button to see the result of control solution measurement.

3-1) Press the LEVEL1 or LEVEL2 button to see only the result of that LEVEL. Press " Print " button to re-print the selected result.

www.greencrossms.com

Hemoglobin A1c Analyzer

11. MANAGING GLUCOSE

	NGSP	IFCC
Pre-diabetes	5.7~6.4%	39~46 mmol/mol
Diabetes	≥6.5%	≥48 mmol/mol
Diabetic patient care goals	< 7.0%	< 53 mmol/mol

Source : American Diabetes Association Clinical Practice Recommendation (2010.01)

IFCC : International Federation of Clinical Chemistry

NGSP: National Glycohemoglobin Standardization Program

12. TRANSFERRING DATA

It transfers the result data from The GREENCARE A1c to a PC by a "USB A to B type" cable that the extra program provides user with usability. The extra PC program can be change the setting value of analyzer and convert the received data to Microsoft Excel format.

Since GREENCARE A1c transmits measurement results via an Ethernet cable, it can receive measurement results from a specific system used in hospitals, etc. without a PC program, and can be used for various purposes.

13. CARING FOR THE ANALYZER

To avoid contaminating your test analyzer and test cassette, please wash and dry your hands thoroughly before use.

13-1. Cleaning and disinfection

Cleaning and disinfecting your analyzer is important to prevent the spread of infectious diseases. Clean your device with soft and dry fabric, and make sure the device is clean all time.

13-2. Storage and handing

1) Test Analyzer

- Store test analyzer at -10°C ~60°C
- Do not leave the device at dusty area.
- Do not leave the device where humidity or temperature changes dramatically.
- Do not leave the device where strong magnetic field occurs.
- Avoid impact.

2) Test Kit

- Store Test Kit at : 2°C~ 30°C
- Store Test Kit at: 15~75% (relative humidity)
- Do not leave the Test Kit where humidity or temperature changes dramatically.
- Use the Test Kit immediately after unsealing it from the package.

3) Check Kit

- Store Check Kit at -10°C~60°C
- Keep away from direct sunlight.
- Keep in the pouch.
- Do not leave the Check Kit where humidity or temperature changes dramatically.
- Do not damage the Check Kit information label.
- Do not damage the Check Kit measurement window.
- The validity period is three years from the date of manufacture (marked in the Check Kit).

Hemoglobin A1c Analyzer

13-3. Checking the analyzer

1) To measure HbA1c, press the cover button of the analyzer.

- 2) Insert the Check Kit into the cassette holder when "Insert the test cassette" message is displayed. Please insert the Check Kit at a stroke. If it does not proceed to the next step, press the " button to proceed to the next step.
- 2-1) If the Check Kit code is not recognized correctly, a screen appears asking if you have used the test cassette. if pressing "Test Cassette?" on the screen, "Check Kit?" screen appears. after press the "OK"" button to proceed with the cassette code entry step.
- 2-2) Select the " < or " < or
- 3) After the Check Kit is normally inserted, the "Close the cover" indication is displayed. The test starts automatically once the cover is closed.

4) The rest of test time is displayed if the cover is closed.

www.greencrossms.com

Hemoglobin A1c Analyzer

5) After the test is done, the result popup is displayed. After pressing "

6) Open the cover to remove the Check Kit.

7) Take out the Check Kit from the analyzer cassette holder by pulling out.

13-4. Error message and troubleshooting

MESSAGE	MEANING	ACTION
Error Cassette code	The cassette code has been entered incorrectly.	Please click the error message. Remove the cas- sette, and close the cover. Perform the test again. If the error message persists, contact your local representative.
Error Cassette Expiry date	The date of test cassette is expired.	Please click the error message. Open the cover and take out the test cassette. Make sure the avail- able period. Set a date if the date of analyzer is not the present date. If there are no problems with the available period and the date of analyzer, validate the analyzer with a Check Kit. If the error message persists, contact your local representative.
Error Cassette insert	Problem with cassette recognition sen- sor.	Please remove the test cassette. Check the sensor status in setting menu. If the error message per- sists, contact your local representative.

MESSAGE	MEANING	ACTION
Error Cover Open	Opening the cover while test or check is in process.	Please close the cover, and click the error mes- sage. Open the cover and remove the cassette. If this error message is alert with the closed cover, check the sensor status in setting menu. If the error message persists, contact your local repre- sentative.
Error Date/Time	Problem with date/time in analyzer.	Please click the error message. Check the day and time initialized in analyzer. (2000/00/00/00) Set the present day and time in setting menu. The power switch of analyzer is off and on again. If the error message persists, contact your local representative.
Error Warm up	Problem with Warm-up.	Please chick the error message. Power off the analyzer. After about 10 minutes, power on it. If the error message persists, contact your local representative.
Error Ligh Variation	High variation of measured values.	Please click the error message. Keep away from the direct sunlight. The analyzer is validated by a Check Kit. If the error message persists, contact your local representative.
Error Ununiformity	Problem with ununiformity of measure- ment window.	Please click the error message. Open the cover and remove the cassette. Insert a new test cas- sette and perform the test again. If the error mes- sage persists, contact your local representative.
Error Excessive blood	Excessive blood error.	Please click the error message. Open the cover and remove the cassette. Re-test the blood sample $(3.5 \mu l)$.
Error Less blood	Less blood error.	Please click the error message. Open the cover and remove the cassette. Re-test the blood sam- ple $(3.5\mu \ell)$.

MESSAGE	MEANING	ACTION
Error Operating Position	Problem with operating position.	Please click the error message. Remove the cas- sette if it is. Use the Check Kit to validate the ana- lyzer and perform the test again. If the error mes- sage persists, contact your local representative.
Error Printer paper	No printer paper.	Please click the error message. Check the cover of printer to be closed, and the paper roll to be. If there is another problem, contact your local rep- resentative.
Error Reagent pack sensing	Problem with reagent pack recognition.	Please remove the test cassette. Check the sensor status in setting menu. If the error message per- sists, contact your local representative.
Error Reagent pack State	Problem with reagent pack. The seal of reagent is not removed.	Please click the error message. Open the cover and remove the cassette. Insert a new test cas- sette and perform the test again.
Error Reagent pack insert	The cassette is left in the analyzer for an extended period of time.	Please click the error message. Remove the cas- sette and use a new cassette and reagent pack.
Error Thermistor temperature	Problem with ambient temperature sensor.	Please click the error message. Power switch off and on again. If the error message persists, con- tact your local representative.
Error Temperature is too high	The ambient temperature is too high.	Please make sure the analyzer is within normal operating temperatures for at least 30 minutes at power on and re-test.
Error Temperature is too low	The ambient temperature is too low.	Please make sure the analyzer is within normal operating temperatures for at least 30 minutes at power on and re-test.

MESSAGE	MEANING	ACTION
Error HbA1c is too high	The HbA1c test result is higher than 15%.	Please remove the test cassette. Insert a new test cassette and perform the test again.
Error HbA1c is too low	The HbA1c test result is lower than 4%.	Please remove the test cassette. Insert a new test cassette and perform the test again.
Error Parameter invalid	Problem with setting value recognition in analyzer.	Please click the error message. Power switch off and on again. If the error message persists, con- tact your local representative.
Error Check Kit	Problem with the analyzer or Check Kit.	Please click the error message. Check the Check Kit measurement window for damage. Validate the analyzer with a Check Kit again. If the error message persists, contact your local representa- tive.
Error Reuse reagent pack	Problem with reusing the reagent pack	Please click the error message. Remove the test cassette. Insert a new reagent pack and perform the test again.

Hemoglobin A1c Analyzer

14. SPECIFICATION

Product name	GREENCARE A1c Analyzer	Memory	400results
Model	RT-100	Weight	1,600g
Calibration	Spectroscopy	Dimensions	205(W)x200(L)x150(H) mm
Sample material	Capillary whole blood. Venus blood with antico- agulant.	Package count	1 Set
Sample size	3.5 µl	Display	4.3inch LCD (480x272 Pixel)
Hematocrit	25~60%	Power	Input: 100-240 VAC, 50-60Hz Output: 12 VDC, 3.33A
Test time	About 5 minutes	Operating humidity	15~75%
Measurement Range	HbA1c: 4.0 ~ 15.0 % (20 ~ 140 mmol/mol) eAG: 68 ~ 384 mg/dL (3.8 ~ 21.3 mmol/L)	Operating temperature	17~32℃
Measurement Unit	% or mmol/mol	Storage temperature	-10~60°C
Storage humidity	15~75%		

15. PERFORMANCE

15-1. ACCURACY

This is measured by Tosoh HLC-723 GHb G7 as reference equipment with GREENCARE A1c Hemoglobin A1c Test Kit.

	Acceptance ratio
Within ± 3%	50%(20/40)
Within±6%	92.5%(37/40)
Within ± 10%	100% (40/40)

15-2. PRECISION

Within Run Precision

HbA1c Concentration (%)	5.7%	9.3 %
Mean(%)	5.5	8.9
STD(%)	0.1	0.2
CV(%)	2.4	1.9

Hemoglobin A1c Analyzer

Day to day Precision

HbA1c Concentration (%)	5.7%	9.3 %
Mean(%)	5.4	9.0
STD(%)	0.1	0.2
CV(%)	2.6	1.9

16. WARRANTY

This product has qualified the internal quality inspection. Make sure you bring the product warranty for the product repair service.

Model name	RT-100	Name of buyer	
Serial number		Phone number	
Date of purchase		Address	
Purchased from		E-mail	
Warranty period	2 years		

** Repair or replacement of product applies only to the original buyer in possession of this product warranty. Please keep this document for the warranty. You must write down the date of purchase. If you have questions or need more information, please contact service center.

Hemoglobin A1c Analyzer

17. SYMBOLS

Symbols	Description	
Caution	May cause product damage or loss of data	
Warning	May cause personal injury, or inconvenience to patient due to false test results/diagnosis	
Ŕ	Biological risks	
	Consult Instruction for Use	
IVD	In vitro diagnostic medical device	
	Manufacturer	
~~~	Date of Manufacture	
X	Waste Electrical and Electronic Equipment (WEEE)	
	Direct current	
LOT	Batch Code	
SN	Serial Number	
EC REP	Authorized Representative in the European Community	
CE	This product fulfills the requirements of Directive 98/79/EC on in vitro diagnostic medical devices	

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www.greencrossms.com

#### Model.RT-100

GREENCARE A1c analyzer is an IVD device for measuring

the hemoglobin A1c in both capillary

whole blood and venous blood.

Manufacturer GREEN CROSS MEDICAL SCIENCE

26, Mugeuk-ro 65beon-gil, Geumwang-eup, Eumseong-gun, Chungcheongbuk-do, Republic of Korea

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MT ProMedt Consulting GmbH Altenhofstrasse 80, 66386 St. Ingbert, Germany